

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your dealer.



Drive Medical Ltd

Ainley's Industrial Estate, Elland HX5 9JP.

www.drivemedical.co.uk

Document code: R002_IFU

Channel ramps

A large, vertical version of the Drive Medical Ltd logo, rendered in white on a dark blue background. The word "drive" is written in a bold, lowercase sans-serif font, with a trademark symbol (TM) at the top right. The logo is oriented vertically, reading from bottom to top.

channel ramps

Drive channel ramps provide a portable and lightweight solution to some access problems experienced by wheelchair users.

The ramps can be used to obtain access to motor vehicles, buildings, etc. They have a weight limit of 270kg (145kg per ramp).

TO EXTEND:

1. Slowly pull out the narrowest section of the ramp whilst holding the widest section, until it is locked in position.
2. To extend the ramp to its full length, continue to pull the narrowest section of the ramp whilst pressing button 1. Do this until the section is locked in position.
3. Repeat this for the other ramp.

TO RETRACT:

1. Position the narrowest section into the main body of the ramp. Do this by pressing button 2 whilst pushing the narrow section into the body of the ramp.
2. To complete the retraction operation, continue to push the narrowest section into the body of the ramp whilst pressing button 3. Do this until the ramp is locked into position.
3. Repeat this for the other ramp.

using the channel ramps

- Always fully extend the ramps before use to minimise the risk of injury, and also make any ascent or descent using the ramps easier.
- Select a place to use where the ramps have the minimum gradient possible. Ensure the gradient of the ramps is not beyond the ability of the wheelchair or occupant.
- Place the wide end of the ramp on the ground or low point, and place the narrow end of the ramp on the vehicle or high ground.
- Ensure the ramps are safe before use.
- Ensure the ramps run parallel to each other before use.

safety precautions

- Always use both ramps, and ensure they are level and parallel with each other.
- Ensure the ramps are securely locked to their extents.
- Do not attempt to use the ramps on gradients of 1 in 8 or steeper. Please also check the gradient your wheelchair or scooter is capable of.
- Do not exceed the weight capacity of the ramps. Only allow one person / mobility vehicle to use the ramps at once.
- Use the straps included to wrap around the ramps lengthways when folded. These secure the ramps in transit and when handling, otherwise the ramps can involuntarily unfold.